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Cleaning/Disinfection:

Any impurities that are recognizable should be removed immediately to avoid drying. Cleaning is best done with a damp cloth and mild plastic cleaner or soapy liquor. Do not use oily or fatty substances (risk of slipping!) or sharp abrasives. If necessary, the suction handle can be treated with a commercially available, chlorine-free device disinfectant in the wipe sanitizer procedure. Important: please observe the manufacturer's application regulations.

⚠ Safety instructions

- Note the maximum load capacity (60 kg)
- Do not strain the handle with your complete body weight
- Use the suction handle exclusively for purposes described above
- Before each use, check the sufficient adhesion of the handle, install it again, if necessary, in accordance with the user manual
- Do not permanently expose the handle to direct sunlight so as not to damage the rubber of the suction cups
- Defective, damaged or improperly installed products must not be used, please consult your reseller
- Use only original spare parts
- Let the handle only come into contact with uninjured skin!

Warranty

According to the statutory warranty claims. Excluded from the guarantee are defects resulting out of natural wear and tear, intent, negligence or improper handling or use and defects which affect the value or usability of the tool only insignificantly.

We reserve the right of modifications serving the technical improvement.


Suction Cup Grab Bar / Telescopic Suction Cup Grab Bar

Item no. 17 350 / Length: 32 cm


Item no. 17 362 / telescopic from 56-67 cm

Attachment with vacuum lever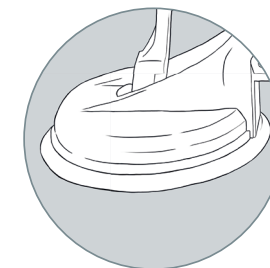

Rubber suction cup: diameter 12 cm

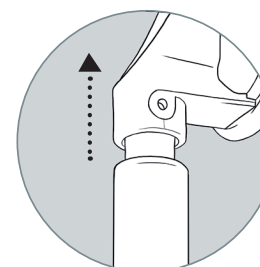Telescopic Suction Cup Grab Bar: Adjustable from 56 to 67 cm

Sundo Homecare GmbH,
Bahnhofstraße 13, DE-30916 Isernhagen, www.sundo-homecare.de

H&K International Trade(Trading) Limited, 8F, Building 5, Jinhua Wanda Plaza,
No.107 Guangnan Road, Jindong District, Jinhua, Zhejiang, PR China

EC REP Luxus Lebenswelt GmbH, Kochstr. 1, DE- 47877 Willich

► Suction Cup Grab Bar / Telescopic Suction Cup Grab Bar

Intended Use

The Suction Cup Grab Bar is a medical device. It is primarily used for safe hold in the bathroom, shower and toilet area, but can also be used in all other rooms. It can be installed on smooth surfaces without tools and dismantled without residue.

Indication

Disease-related loss of strength/functional impairment of the lower extremities with sufficient functioning of the upper extremities, gait and stand-up uncertainty

Contraindication

Insufficient standing capacity, dizziness, insufficient physical and/or mental ability for use

Description

Suction bars facilitate or support the independent sitting down and getting up or holding tight. With the help of the handles, people who still have mobility and strength, but already need support, can continue to use washbasins, showers, toilets etc. independently. Please note that the suction handles are not suitable to carry the entire weight of a person (max. load 60 kg). For this purpose please select other handles that can be screwed to the wall. The bar must only come into contact with intact, uninjured skin. Any use other than that mentioned above is not intended. Sundo GmbH is not liable for the resulting damages and improper assembly, the risk is borne exclusively by the user. In the event of serious incidents related to this product, please inform the manufacturer or the competent authority.

Versions

Two different lengths are available (one adjustable in length), the appropriate selection is the responsibility of the specialist staff and depends on the needs and the location.

Before use / Maintenance

First, read the present instruction for use completely and keep it together with the packaging until the end of the product life. Please check if the product is undamaged before first and any use by visual inspection. Before each use, also check the secure attachment of the handle. If the liability is not given or is not sufficient, please loosen the handle according to the assembly instructions and reattach it. Check the suction plates regularly for wear. In case of signs of damage or wear, the handle must not be used (any more). No further maintenance is required.

Assembly

The product must be assembled by authorized, trained personnel. First, select a suitable location for the Suction Cup Grab Bar. The surface must be smooth, dry, grease-free and clean. The suction plate must be able to lie completely on the surface without, for example, a tile joint or similar passing through the support surface. The suction plates should be attached at least 1.5 cm away from joints or edges. To do this, first carefully and thoroughly clean the surface intended for assembly and the suction plates of the handle. The fastening levers point inwards towards the bar. Now place the Suction Cup Grab Bar on the selected mounting location and press the mounting levers outwards to fix the handle. Due to the resulting vacuum, the lever can only be pushed down with a certain amount of effort. The greater the resistance, the better the adhesion of the handle will be. If there is insufficient liability, the handle must not be used, but must be loosened and reattached!

Addition to item no. 17362, Telescopic Suction Cup Grab Bar

Remove the individual parts from the carton and connect the suction plates to the bar by pushing the handle part into the suction plate's receiving tube. Make sure that the silver button in the designated opening is audibly, visibly and noticeably snapped in / appears. Proceed in the same way with the second suction plate. The length of the handle can be determined by pulling out or pushing together and is preserved after attachment.

Dismantling

Push the mounting lever of the suction handle back towards the holding rod. Then carefully slide your fingertips under the suction plate and lift it. Do the same with the second suction plate. Attention, to prevent the handle from falling hold it firmly at the same time.

Technical data

Item no. 17 350 / Suction Cup Grab Bar Item no. 17 362 / Telescopic Suction Cup Grab Bar			
Material:	ABS, rubber, stainless steel	Handle distance to the wall:	approx. 3,5 cm
Color:	white	Lifespan:	approx. 5 years
Total length:	17 350 32 cm 17 362 56 - 67 cm		
Grip area:	approx. 30-44 cm		