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Instructions for use

EN

Read the complete instructions for use carefully before assembling, adjusting or using the product for the first time. Keep the instructions in a safe place so that they are available for later use.

Intended purpose

The commode chair is used for hygienic toilet use and to facilitate personal care for people who are in need of care and/or have limited mobility.

Indication

The use of the product is suitable for people with

- slight loss of balance
- slight loss of strength
- limited mobility of the legs, knee joints and upper limbs

Contraindication

The use of the product is unsuitable for persons with

- severe sensory disturbances
- severe loss of balance
- severe loss of strength
- severe pain in the trunk or in the upper and lower limbs

Scope of delivery

Shower and toilet chair Atlantic incl. backrest and seat with cover
Bucket including lid
Instructions for use

Technical data

Weight: 13 kg
Height: 92 cm - 100 cm
Width: 45 cm
Depth: 82.5 cm
Seat height: 48.5 cm - 56 cm
Armrest height: 71 cm - 79 cm
Seat surface: 43 cm x 39 cm
Backrest: 44 cm x 18 cm
Footrests: approx. 15 cm x 14 cm
Distance between seat/backrest: approx. 22 cm
Max. load: 136 kg
Material: Powder-coated aluminium, PU, thermoplastic elastomer

Assembly

1. The shower and commode chair is not supplied pre-assembled in a cardboard box. Check the contents and packaging for integrity and completeness.

You will receive the following individual parts: 1x frame of the chair, 4x wheels including insertion tubes and clamps, 2x footrests including insertion tubes, fixing screws and nuts, 1x backrest including 2 insertion tubes and fixing screws, seat with cover, bucket with lid.

2. Remove the frame from the box and place it on the floor using the rubber feet. Attach the padded toilet seat using the fixing clips.

3. Now attach the four wheels using the clips. It is important that all four wheels are set at the same height.

You can now slide the bucket (optionally with lid) into the holder from behind.

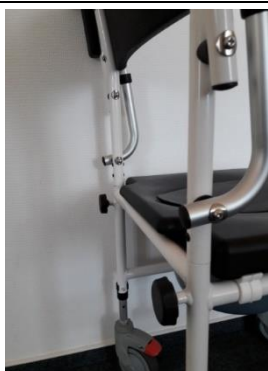
4. Press down the red brake levers on the wheels so that the frame stands firmly on the ground. Insert the tubes with the footrests into the openings provided on the right and left. Ensure that the black bracket runs along the tube. Turn the footrests forwards until the pin engages in the bracket. The footrests point inwards. Fold the footrests down for use.

5. To adjust the height of the footrests, loosen the wing nut, remove the screw and adjust the tube to the desired position. Then reinsert the screw and tighten the wing nut.

6. Insert the lower tube ends of the backrest into the openings on the back of the frame and secure them with the two button screws. Fold the armrests down.

7. To release the brakes, press the black catch.

Safety Instructions

- Before each use, check that the Atlantic shower and commode chair is undamaged. Do not use a defective shower and commode chair!
- Damaged or worn parts may only be replaced by a specialist dealer!
- Make sure that the brakes are locked before anyone sits on the Atlantic shower and commode chair.
- Before each use, check that the backrest and footrests are secured.
- Make sure that the wheels and footrests are set at the same height. Ask your specialist dealer or care personnel to assist you with the adjustments.
- Before use, check that the bucket is firmly seated in the holder and that the cover is flush with the edge of the toilet seat.
- Make sure that the armrests are folded down during use. There is a risk of jamming.
- Only fold down the footrests once the person has taken a seat in the shower and commode chair. There is a risk of tripping and injury.
- Do not stand on the footrests and do not lean on one side of the armrests. There is a risk of falling and tipping over.
- Do not stand on the Atlantic shower and commode chair and do not leave children unattended near it. Risk of injury.
- The maximum seat load is 136 kg.

Cleaning and disinfection

The product should be cleaned after each use and disinfected regularly. This is absolutely necessary if it is used/reused by another person. To clean, rinse the product with warm water and use mild soapy water for cleaning. Dry the product with a soft cloth and ensure good ventilation in the bathroom. Disinfection can be carried out with a commercially available disinfectant. The application instructions provided by the manufacturer must be observed. Do not use abrasive or caustic cleaning agents or high-pressure or steam cleaners!

Maintenance

The product is maintenance-free. If you notice any damage, signs of wear and/or loosening of the mounted parts, contact your specialist dealer.

Re-use

The product is suitable for reuse. Before passing the product on to another user, it must be checked, cleaned, disinfected and serviced by a specialist dealer. If parts of the product are damaged or destroyed, they must be replaced with new ones. All loosened screws must be tightened. The product must be in perfect condition before being used again. It is forbidden to reuse the product if irreparable damage to the product is detected!

The manual must be included with every change of user.

Disposal

Disposal options may vary depending on local regulations. Check with your local waste disposal centre whether the product can be disposed of as normal household waste or whether there are separate regulations for disposal.

Storage

During use in the bathroom, good ventilation must be ensured to prevent the formation of deposits. The product must be dried after use. To store the product, it must be completely dry. Then store it in the polybag and cardboard box provided in a dry place protected from light.

Service Life

If the product is used for its intended purpose and cared for, checked and maintained in accordance with the specifications, a service life of three years can be assumed. However, the service life can vary considerably depending on the handling and care of the product. For example, particularly careful handling of the product can extend the service life, while extreme and improper handling of the product can shorten the service life. The determination of the service life by Rehaforum Medical GmbH does not constitute an additional guarantee.

Warranty

The warranty period is 24 months and applies to all product defects that are demonstrably attributable to manufacturing or material defects. Excluded from the warranty are Damage caused by natural wear and tear or intent. Furthermore, the warranty does not cover damage caused by incorrect cleaning and care products or damage caused by negligent and improper operation or use.

Notification / Incident

Any serious incident that has occurred in relation to the device must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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Rehaforum Medical GmbH, Daimlerstraße 12a, DE-25337 Elmshorn